

# HUR5 Cavitation Operate Manual

User manual



Warning	1
Parts introduction	2
Annexes	2
Opearation	3
Eyes care	3
Facial care	3
Body care–de-fatting	4
Body care–Thin body styling	4
Others	4
Note	5
Forbidden group	5
Technique parameters	6
Annex List	6

---


## Warning

Dear user:

Thanks for using our company's products, in order to fully make use of the products, we sincerely suggests follow:


- ◆◆ Please read this instruction in detail, and keep it well for further reading and consulting.
- ◆◆ Please follow the instruction to fit and operate the machine correctly.
- ◆◆ Please do not remove and change any accessories of the machine.
- ◆◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆ It forbids others to change and maintain the machine except the professional people.
- ◆◆ Please contact us if there is any trouble during operation.

## Parts introduction

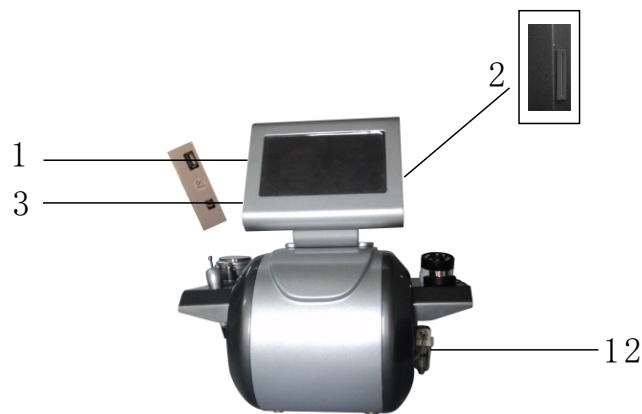

1. USB outlet (for multimedia function)
2. IC card socket
3. Earphone jack
4. 2 polar operating socket
5. 3 polar operating head socket
6. M38 high-frequency waves operating head socket
7. 8 polar vacuum operation head jack
8. Foot switch jack
9. M80 ultrasonic head jack
10. General power switch
11. power line jack

## Accessories

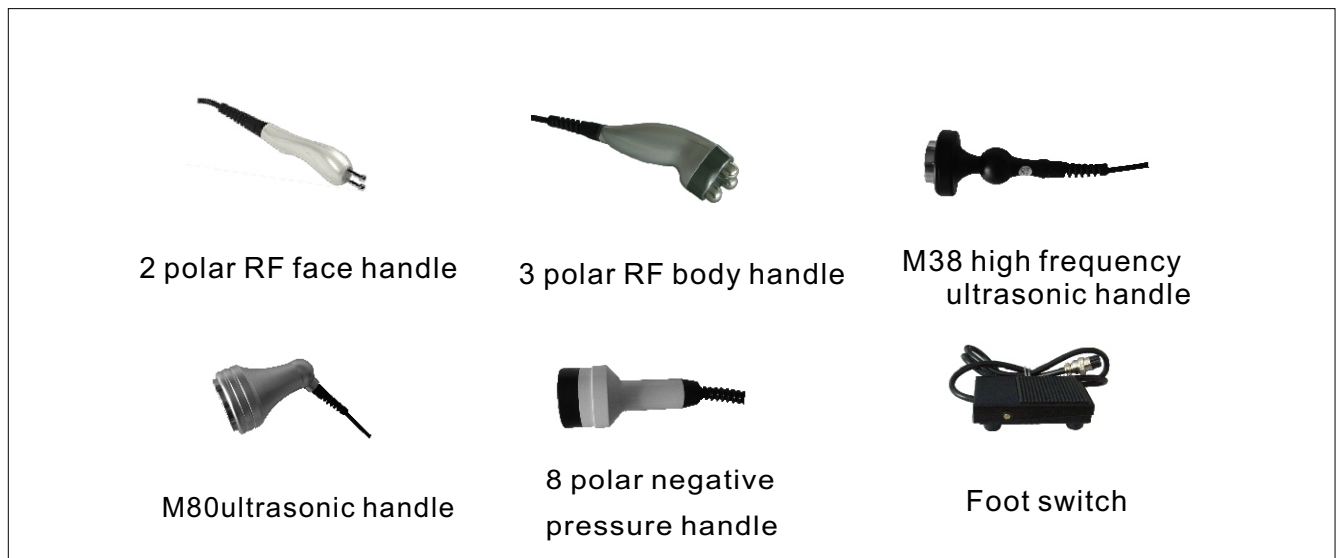


## Operation

1. Connect the accessories to the host.

- ① Insert RF 2 polar operating head to [4]
- ② Insert RF 2 polar operating head to [5]
- ③ Insert M38 high-frequency waves operating head to [6].
- ④ Insert 8 polar vacuum operating Head to [7].
- ⑤ Insert foot switch to [8].
- ⑥ Insert M80 ultrasonic to [9]

2. Insert power line to [11], open the General power switch to [10], enter interface as (Fig1).

3. Insert IC card correctly, otherwise enter interface as (Fig2).

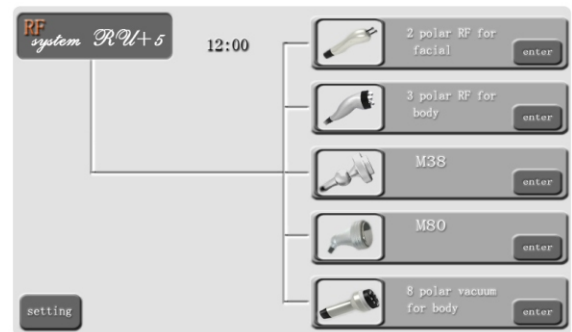

( Fig 1 )

## Face treatment

1. In interface such as ( Fig 1 ), click the "2 polar for face" to enter interface as ( Fig 3 ).


2. click " ← " and " → " regulate output intensity of RF: 1 (weak) ~ 3 (strong).

3. click on " ← " and " → " to set working time, facial care about 20-30 minutes.

4. Click on "START" begin to work, click " pause" stop to work.

5. Apply gel on face around skin, move in circle, the electrode probe move from inside out, operating on both side, right and left, one side over to start to other side, and must lower the intensity accordingly as less fat on forehead and eyes. do not let gel into eyes .

6. When it finished click "EXIT".

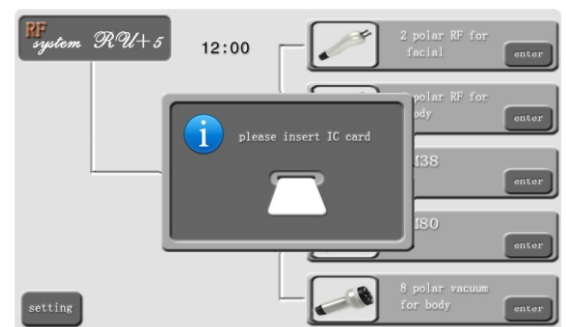

( Fig2 )



( Fig3 )

## Body treatment

1. Click "tri-polar RF for body" in interface as ( Fig 1 ), enter interface as ( Fig 4 ).

2. Adjust output intensity: 1 (weak) ~ 10 (strong).

3. Setting treatment time about 20~30 minutes for each part.

4. Click "start" to work, click "pause" to stop.

5. Put enough gel on treatment body part recommended high intensity on fat stock up part.


6. When it finished, click "EXIT".



( Fig4 )

## M38 slimming operation

- 1.as the screen shows (Fig1),click the “M38” key and get into the work as (Fig5) .
- 2.choose the working mode: P1~P9 (pulse),P10(last)
- 3.adjusttheoutputintensify: 0.5W(weak)~5.0W(strength)
- 4.adjust the photon mode: synchronous/turn off
- 5.Click “Time”on the screen and set working time (20 min every part),.
- 6.click the”start”key to start the machine.
- 7.clean the parts and applied gel on fleshy parts(eg. gel or essence) .The amount depends on moving flexibility of the ultrasonic slimming head.Beautician hold the ultrasound head to move slowly on skin and do not stay unmoved on any part of your body while using ultrasonic cavitation slimming head
- 8.adjust the needed mode, operational time and output intensify according to the treatment. After finish, click the “exit”.
- 9..Clean the leftovers with hot towel, use warm towel to clean the ultrasonic cavitation slimming head.

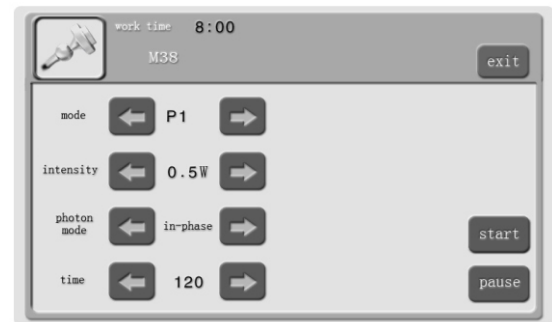

(Fig5)



(Fig6)

## M80 slimming operation

- 1.as the screen shows (picture 1),click the “M80” key and get into the work as the picture 6 shows
- 2.choose the working model:P1~P4 (pulse),P5(last)
- 3.adjust the output intensify:10W(weak)~50W(strength)
- 4.Click “Time”on the screen and set the working time (10~15 min every part), then applied gel on fleshy parts(eg.abdomen,hip,thigh) The amount depends on moving flexibility of the ultrasonic slimming head.
- 5.Click”start”, put the foot switch under your feet, step on to start working, move away for pause.
- 6.Beautician hold the ultrasound head to move slowly on skin, in circle or beeline repeatedly, the other hand push the fat towards the ultrasonic cavitation slimming head.
- 7.Do not use on back, avoid using on bones.
- 8.Do not use on uterus part when women do abdomen care. Abdomen treatment during menstruate should be avoided.
- 9.The machine will pause automatically if the setting time is over.
- 10.Clean the leftovers with hot towel, use warm towel to clean the ultrasonic cavitation slimming head.


## Use RF caution

- 1.Applied enough gel on skin need treated.
- 2.Do not stay unmoved on any part of your body while using ultrasonic cavitation slimming head, do not stay on bones and used in the human body heart throat and eyelids. If the skin is sensitive or becomes unhealthy, the strength of the power should be adjusted down.


- 3.The daily treatment should be within 30 minutes.
- 4.There will be ultrasonic only when you step on the foot switch.
- 5.Do not make the RF electrode stay on any part without moving.The electrode can not allowed to be placed under the state of no-load to avoid the damage caused by high heat.

## 8 polar vacuum

- 1.Click 8 polar “RF”vacuum,enter interface as (Fig 7).
- 2.Adjust output intensity:0(weak)~10(strong).
- 3.Adjust vacuum intensity:
  - 10KPa(weak) ~100KPa(strong).
- 4.Choose vacuum mode as (Fig 8): P1~P6, PE
  - ① P1~P5 vacuum mode has default times,  
Need to adjust inspiratory, releasing air time.
  - ② P6 mode has no adjust function.
  - ③ Can set vacuum mode by yourself (0.1S~3.0S).
  - ④ When finish setting, click return.
- 5.Adjust times (about20minute) .
- 6.Nursing parts daub wipe essential oils or massage oil, click"start".Set time arrived, instruments will be automatically suspended, press any key to continue working.

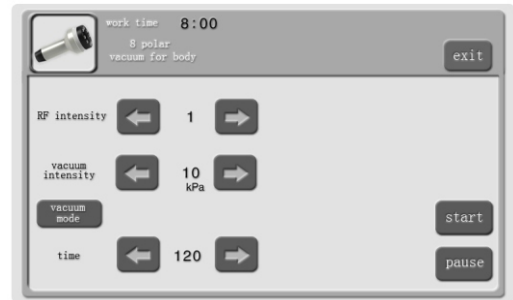

(Fig7)

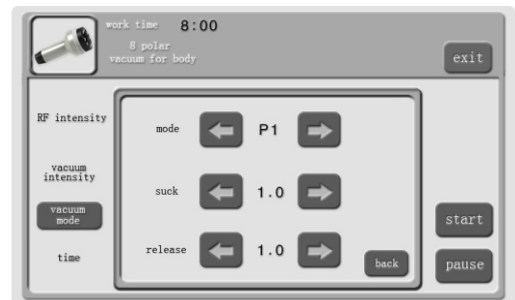

(Fig8)

## Vacuum note


- 1.Need to apply oils or massage oil on the skin as convenient to move; for another, avoid the skin because of dry and feel pain .
- 2.Adjust the negative pressure suction depends on the strength of the guest when, should be determined by weak gradually stronger
- 3.Do not operate on the broken skin .

## Other

Click “setting” in interface as( Fig 1).

- ①Click “multimedia” enter interface as (Fig 7),insert USB to [1],insert headphone to [3].
- ②Click “basic setting”, enter interface as (Fig 8).

**Notes “advanced setting” is for manufacturer**

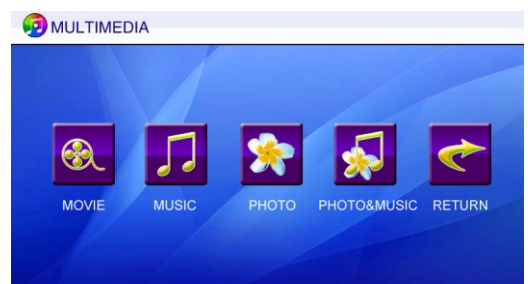

(Fig9)



(Fig10)

---

---

## Caution

- 1.Wear insulated gloves during operation
- 2.Do not wear any aglet, both for the beautician and the users.
- 3.The electrode probe must be fully in contact with skin.
- 4.Do not make the RF electrode stay on any part without moving.keep it moving on skin slowly.
- 5.Skin around eyes and forehead is very thin,the RF output intensity should be relatively weakened.
- 6.Clear the filter cup after use.

**Cleaning Methods:Put some paper at the bottom of the filter bottle,rise the valve with your fingers, let the oil leak.**

## Forbidden group

- 1.People with heart disease or high blood pressure , or configured cardiac pacemaker.
- 2.Patients with acute inflammation, asthma , deep vein thrombosis, thyroncus, cancer.
- 3.People with hemorrhagic disease, trauma or who is bleeding.
- 4.Pregnant women
- 5.Medical Plastic parts, or parts with artificial in filling inside.
- 6.People with metal inside the body(include metallic tooth)
- 7.Patient with skin inflammation or with edema.
- 8.People with an abnormal immune system.
- 9.Numb or insensitive to heat.



---

---

## Technical parameters

Voltage: AC110V/220V/50-60Hz  
Power:  $\leq 100W$   
Output RF: 5MHz  
Output ultrasonic: 40KHz/1MHz  
Negative pressure: 100KPa

## Annex list

1.Host	1set
2.Power line	1pc
3.User manual	1copy
4.Bipolar RF handle for facial	1pc
5.Tripolar RF handle for body	1pc
6.M38 ultrasonic handle for facial	1pc
7.M80 ultrasonic handle	1pc
8.8 polar RF&vacuum	1pc
9.foot switch	1pcs
10.accessory frame	1pc
11.white fast connection sealing ring	5pcs
12.Hexagonal screws	8pc
13.Hexagon wrench	1pc