

# EB20 Multipolar RF System

## Operate Manual



Warning	1
Parts introduction	2
Accessories	2
Operation	3
Eyes around treatment	3
Facial skin treatment	3
Body treatment-fat dissolving	4
Body treatment-slimming&shaping	4
Other	4
Caution	5
Foribben group	5
Parameters	6
Accessories list	6

---


## Warning

Dear user:


Thanks for using our company's products, in order to fully make use of the products, we sincerely suggests follow:

- ◆ ◆ Please read this instruction in detail, and keep it well for further reading and consulting.
- ◆ ◆ Please follow the struction to fit and operate the machine correctly.
- ◆ ◆ Please do not remove and change any accessories of the machine.
- ◆ ◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆ ◆ It forbids others to change and maintain the machine except the professional people.
- ◆ ◆ Please contact us if there is any trouble during operation.

## Parts introduction

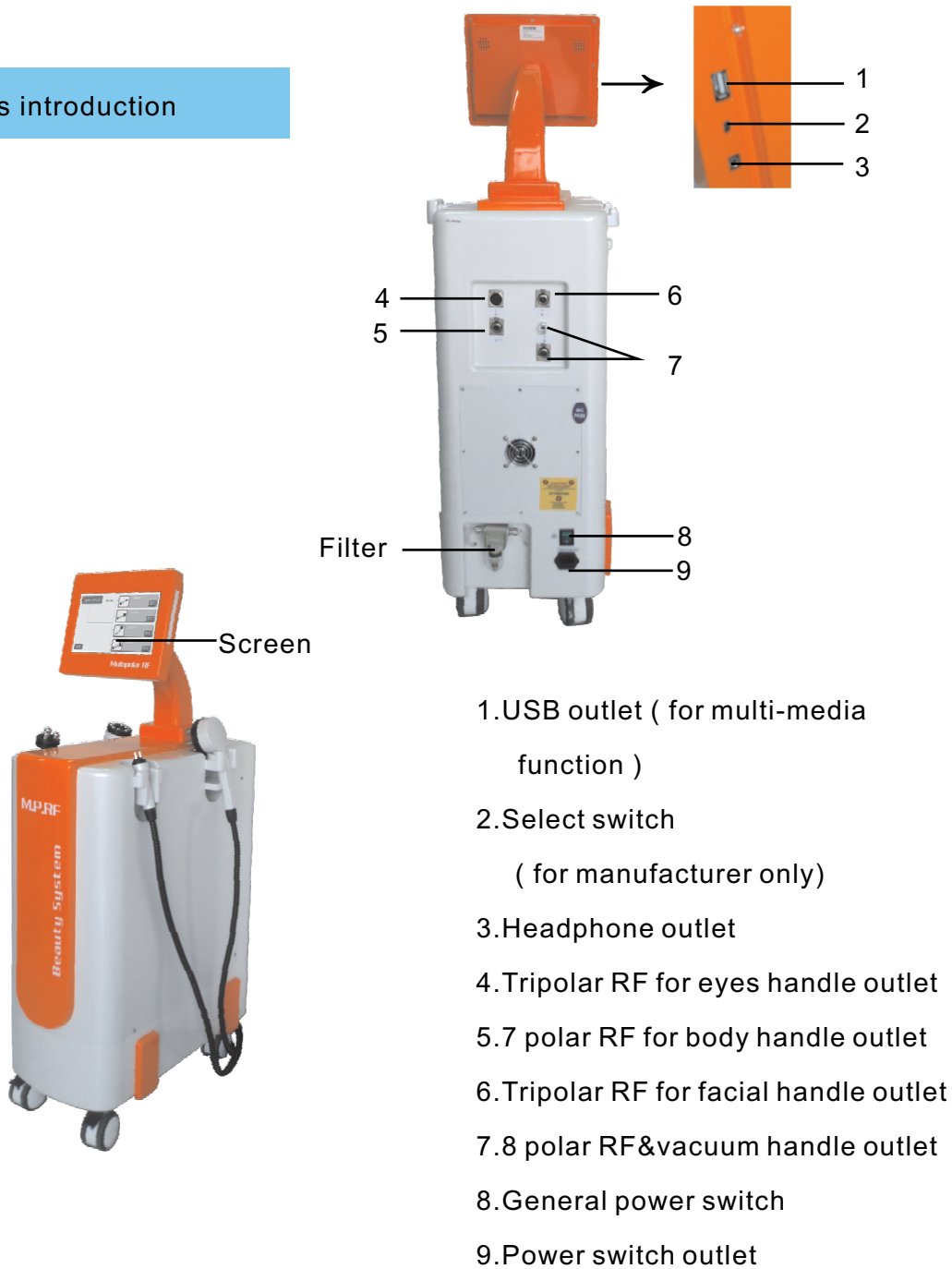

## Accessories

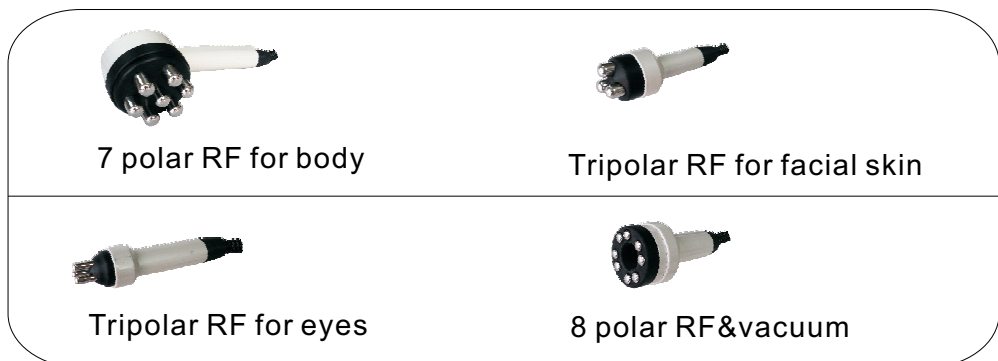

## Operation


- 1.Connect the screen to the host.
- 2.Connect all accessories to the host.
  - ①Insert tripolar RF for eyes handle to [4].
  - ②Insert 7 polar RF for body handle to [5].
  - ③Insert tripolar RF for facial for body handle to [6].
  - ④Insert 8 polar RF&vacuum handle to [7].
- 3.Connect power line [9],turn on general power switch



(Fig 1)

## Eyes around treatment

- 1.Click “tripolar RF for eyes” in interface as Fig 1,ente interface as Fig 2.
- 2.Click “←” “→”to adjust RF intensity :  
1 ( weak ) ~3 ( strong ) 。
- 3.Click “←” “→”to set treatment time, for eyes around about 15~20 minutes.
- 4.Click “start”.
- 5.Apply gel on eyes around skin,move in circle, the electrode probe move from inside out, move to the temple, don't let gel enter eyes. lift up the area of the Canthus and eyebrow, move in line. The eye around skin is very sensitive, so be careful, don't make the probes too hot.dont let gel into eyes.

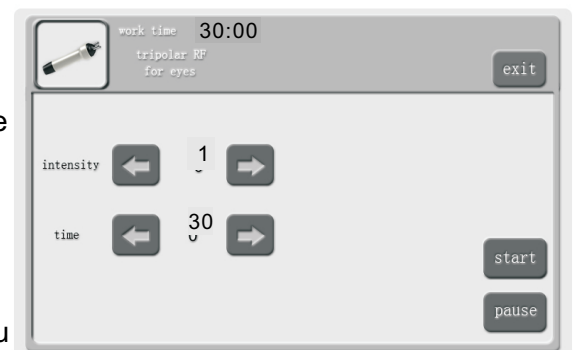

(Fig 2)


## Face treatment

- 1.Click “tripolar RF for facial” in interface as Fig 1, enter interface as Fig 3.
- 2.Adjust output intensity:1 ( weak ) ~5 ( strong ) 。
- 3.Setting treatment time, about 20~30 minutes
- 4.Click “start”.


Divide the facial skin into two part from the middle, apply gel, move the electrode probe from inside out, along the direction of lymph, from the bottom up to do thermal therapy. Do other side in the same way. Repeat for 5~8 times.

The Forehead has less fat, the edge of the eyebrows is also very sensitive, and the output intensity should be lowered.

When it finished, click “EXIT”

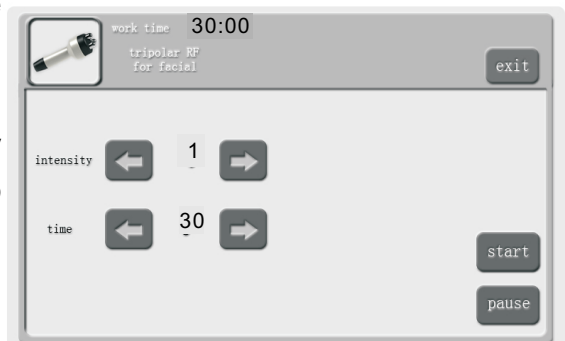

(Fig 3)

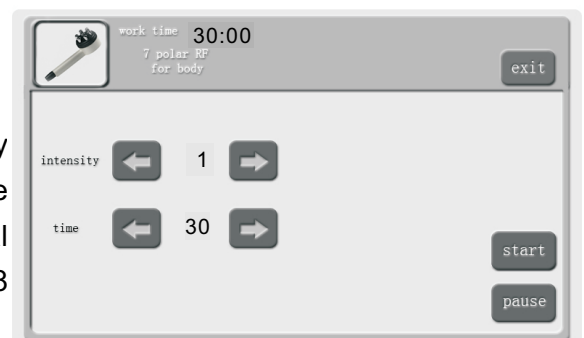

(Fig 4)

## Body treatment-fat dissolving


1. Click "7 polar RF for body", enter interface as Fig 4.
2. Adjust output intensity: 1(weak)~10(strong).
3. Setting working time, about 20~30 minutes.
4. Click "start".
5. Apply gel, If there is much fat, you can increase the output intensity.
6. When the treatment finished, click "Exit". clean the gel on skin.




(Fig 5)

## Body treatment-slimming&shaping

1. Click "8 polar vacuum for body", enter interface as Fig 5.
2. Adjust RF output  
Intensity: 1(weak)~10(strong). Adjust vacuum output intensity: 10(weak)~100(strong). Setting working time, about 20 minutes.
3. Click "vacuum mode", in interface as Fig 6, choose vacuum working mode, mode 1~6.
4. Click "start".
5. Clean skin, apply essential oil, adjust radio frequency intensity, don't make it too strong, just keep it warm.
6. You can change the vacuum working mode, click "pause" before changing.
7. Clean the handle and sterilize after use.

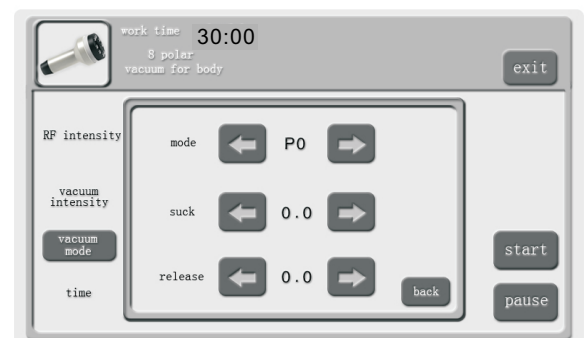

(Fig 6)

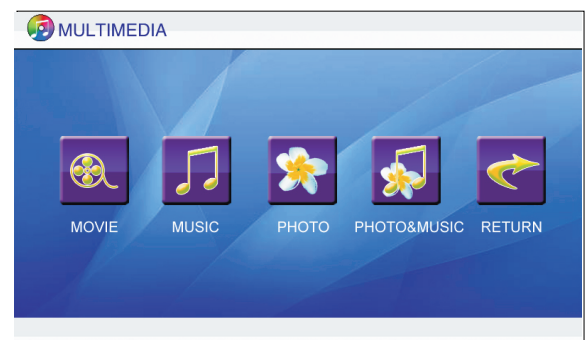

(Fig 7)

## Other

Click "setting" in interface as Fig 1.

- ① Click "multimedia" enter interface as Fig 7, insert USB to [1], insert headphone to [3].
- ② Click "basic setting", enter interface as Fig 8.

**Notes "advanced setting" is for manufacturer only.**



(Fig 8)

---

## Caution

- 1.Wear insulated gloves during operation
- 2.Do not wear any aglet, both for the beautician and the users.
- 3.The electrode probe must be fully in contact with skin.
- 4.Do not make the RF electrode stay on any part without moving.keep it moving on skin slowly.
- 5.Skin around eyes and forehead is very thin,the RF output intensity should be relatively weakened.
- 6.Clear the filter cup after use.

**Cleaning Methods:Put some paper at the bottom of the filter bottle,raise the valve with your fingers, let the oil leak.**

## Forbidden group

- 1.People with heart disease or high blood pressure,or configured cardiac pacemaker.
- 2.Patients with acute inflammation,asthma, deep vein thrombosis, thyroncus,cancer.
- 3.People with hemorrhagic disease, trauma or who is bleeding.
- 4.Pregnant women
- 5.Medical Plastic parts, or parts with artificial infilling inside.
- 6.People with metal inside the body(include metallic tooth)
- 7.Patient with skin inflammation or with edema.
- 8.People with an abnormal immune system.
- 9.Numb or insensitive to heat.

---

---

## Technical parameters

Voltage : AC220V/50Hz

Power :  $\leq 50W$

## Annex list

1.Host	1set
2.Power line	1pc
3.User manual	1copy
4.IC card	2pcs
5.Toucher	1pc
6.Tripolar RF handle for eyes	1pc
7.Tripolar RF handle for facial	1pc
8.7 polar RF handle	1pc
9.8 polar RF&vacuum	1pc
10.Seal ring	5pcs
11.Monitor	1pc
12.Hexagonal screws	4pcs
13.Hexagon wrench	1pc
14.Tutorial CD	1pc